

*Master Challenging*  
**COLONOSCOPIES & ILEOSCOPIES**  
*with Ease*

Endo-Ease Vista® Retrograde Ordering Information

PRODUCT NAME	LENGTH	PRODUCT CODE	QUANTITY
Endo-Ease Vista® Retrograde	100 cm	EEV-200	1 each

*Note: For use with pediatric or slim colonoscopes and enteroscopes with 10.5 mm–11.6 mm O.D.*

**Spirus Medical, Inc.**  
1063 Turnpike Street  
Post Office Box 258  
Stoughton, MA 02072  
U.S.A.

888 297 7220  
SpirusMed.com

**BRIEF STATEMENT**

**INDICATIONS**

The Endo-Ease Vista Retrograde endoscopic overtube is indicated for use with a flexible endoscope to aid endoscopic insertion and advancement during diagnostic and therapeutic lower GI flexible endoscopy.

**CONTRAINDICATIONS**

Upper gastrointestinal procedures. Any indication that would otherwise contraindicate gastrointestinal endoscopy.

**WARNINGS**

The Endo-Ease Vista Retrograde is a sterile device for single use only. Do not autoclave or reprocess.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**CAUTION**

Federal law (USA) restricts this device to sale by or on the order of a physician.

Consult the Instructions For Use for detailed information on the use of this product, indications, contraindications, warnings precautions and potential adverse events.

©2009, Spirus Medical, Inc. All rights reserved.

600-0017 REV. B



# Changing the Boundaries of RETROGRADE ENDOSCOPY

At Spirus Medical, we're advancing endoluminal technology with our commitment to helping you overcome the challenges associated with retrograde GI endoscopy.

**The Endo-Ease Vista® Retrograde** is an innovative over-tube system that helps physicians overcome challenging colonoscopies and enables efficient intubation of the terminal ileum, so ileoscopy can be performed with ease. Pushing the colonoscope through a long, floppy colon can be a difficult task due to excessive looping of the endoscope in this type of anatomy. The Vista's unique spiral helps physicians guide, stabilize and reduce the colon for advanced retrograde procedures.

With an Endo-Ease Vista Retrograde the endoscope can be used to remove large polyps through the Vista, while it remains in place to maintain position in the bowel.

The endoscope can be advanced or removed without altering the position of the overtube. This feature allows the scope to pass through the terminal ileum for performing ileoscopy. The Vista Retrograde acts to stabilize the colon while the endoscope is advanced with little effort.

Discover the benefits of this multifunctional instrument for retrograde endoscopy.

## Vari-flex shaft

- As with conventional flexible endoscopes, the proximal end is stiffer than the distal end.
- Moves with the endoscope and provides a stable channel when you need to advance the scope traditionally.

## Ergonomic grip

- Comfortable foam grip provides an ergonomic surface.

## Gentle-lock coupler


- Connects the Vista Retrograde to the scope.
- Can be quickly engaged and disengaged to allow the scope to move independently while being supported by the Vista Retrograde.
- Will not damage the scope.

## Transition tip


- Soft, beveled end provides a smooth transition from the Vista Retrograde to the endoscope.

## Unique spirals

- Designed to help gather the intestine, maintain position and control withdrawal.
- Assist in advancing the scope through the colon.

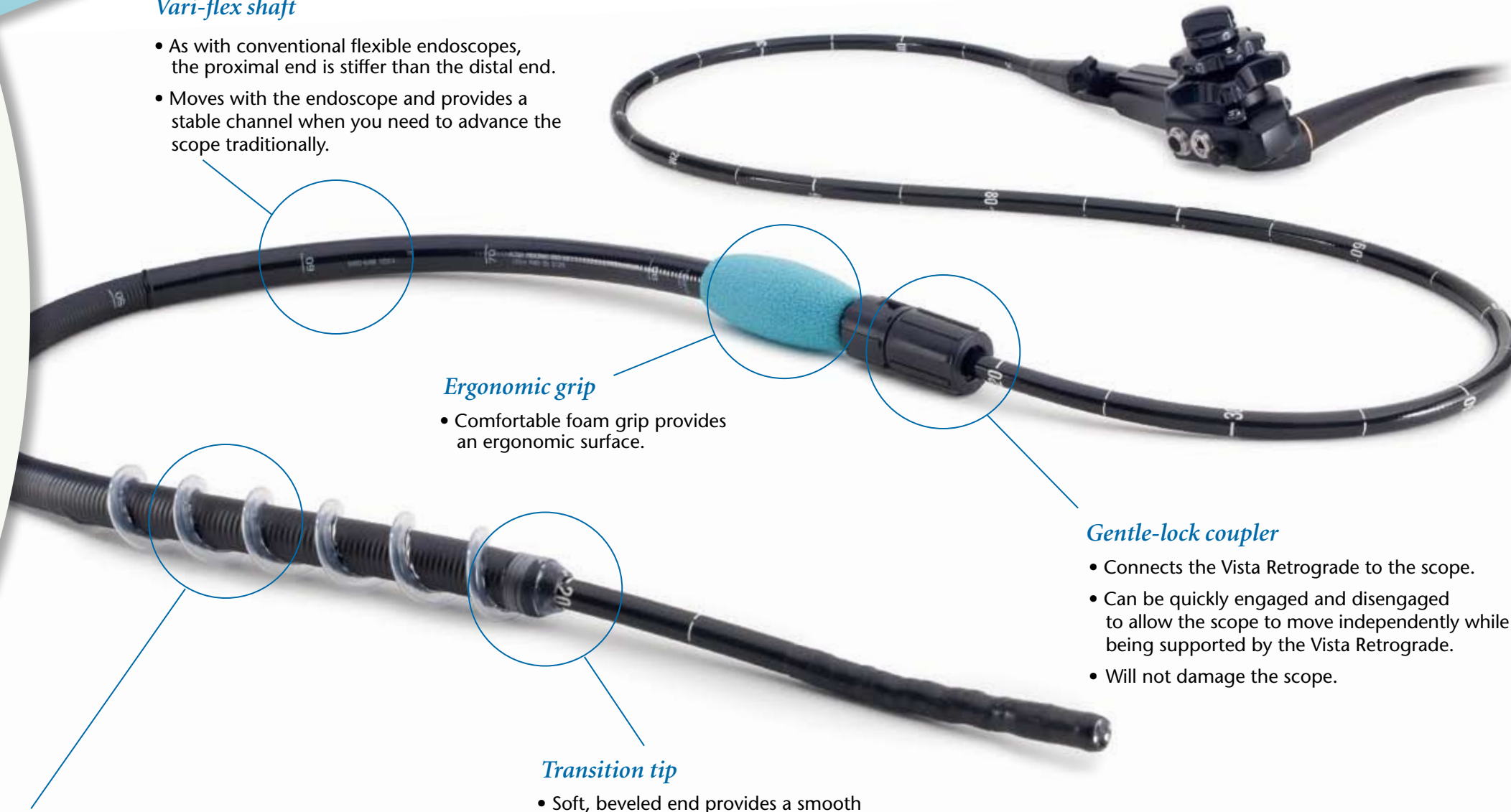

**Endo-Ease**  
**vista**<sup>™</sup>  
retrograde

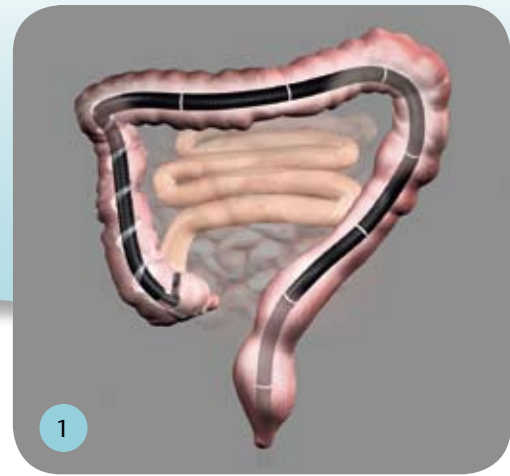

Designed to assist the scope through the colon, the Vista is coupled with your scope, then inserted and advanced into the colon with rotation and gentle forward pressure.



The Vista creates a stable channel through which advanced endoscopic procedures, like ileoscopy, can be performed.



Once into the ileum, standard reduction maneuvers enable inspection and treatment of the small bowel.